Deluxe Instant Ear Thermometer

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Distributed by:
Distribuido por:
Walgreen Co.
Deerfield, IL 60015-4616

Walgreens
The Brand America Trusts®

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Deluxe Instant Ear Thermometer

Instruction Manual

We congratulate you on your purchase of the Walgreens Deluxe Instant Ear Thermometer. The Walgreens Deluxe Instant Ear Thermometer is a high quality product incorporating the latest technology and tested in accordance with international standards. With its unique technology, the thermometer can provide a stable, interference free reading with each measurement. The instrument performs a self-test every time it is switched on to ensure accuracy of measurements.

The Walgreens Deluxe Instant Ear Thermometer is intended for the monitoring of human body temperature in the home. It is appropriate for use on people of all ages.

Please read these instructions carefully before using this instrument and keep it in a safe place.

Table of Contents

- 1. The Advantages of your Walgreens Deluxe Instant Ear Thermometer**
 - 2. Important Safety Instructions**
 - 3. Product Description**
 - 4. How to Measure Body Temperature**
 - 5. How to Read the Display**
 - 6. How to Apply a New Probe Cover**
 - 7. Directions for Use**
 - 8. Changing from Fahrenheit to Celsius and vice-versa**
 - 9. Error Messages**
 - 10. Cleaning and Storage**
 - 11. Replacing the Battery**
 - 12. Warranty**
 - 13. Technical Specifications**
-

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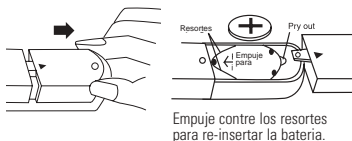
1. Advantages of your Walgreens Deluxe Instant Ear Thermometer

Measurement in 1 second

This innovative technology allows measurement of ear temperature in only 1 second.

Accurate and reliable

Due to the unique patented probe assembly and the advanced sensor, this unit can offer a very accurate and reliable ear temperature measurement.

Gentle and Easy to Use

- Special ergonomic design enables simple and easy use of the thermometer.
- The Walgreens Deluxe Instant Ear Thermometer can be used without interference to daily lifestyle. A measurement can be taken even while a child is sleeping.
- The Walgreens Deluxe Instant Ear Thermometer is less threatening to a child than a rectal thermometer and more pleasant to use than an oral thermometer.

Auto-Display Memory

The product displays the last reading automatically for 2 seconds when the unit is switched ON.

Safe and Hygienic

- No risk of broken glass or mercury ingestion.
- Safe for children.
- Measuring ear temperature with probe covers is the preferred “clean” and hygienic method.

2. Important Safety Instructions

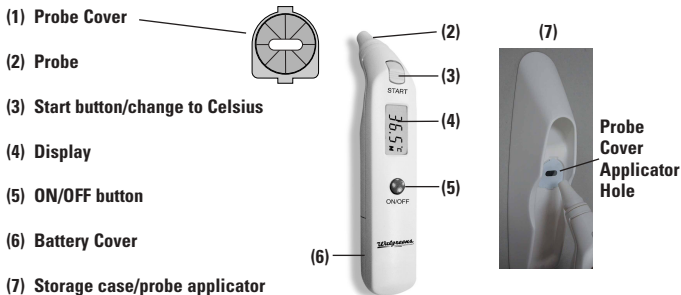
- Never use the thermometer for purposes other than body temperature measurement. Please follow the safety precautions when using on children.
- Always use the thermometer with a new undamaged Probe Cover for each measurement to prevent infection. **Readings taken without probe covers will not be accurate.** Only Walgreens brand probe covers can ensure that you get an accurate measurement from the Deluxe Instant Ear Thermometer.
- **Never immerse the Walgreens Deluxe Instant Ear Thermometer into water or other liquids. (not waterproof) For cleaning and disinfecting please follow the instructions in section 10, “Cleaning and Storage”.**
- Keep the instrument and the probe covers away from direct exposure to the sun and keep it in a dust-free, dry area at a temperature between 10° - 40° (50°F - 104°F).

- Do not use the thermometer if there are signs of damage on the measuring tip or on the instrument itself. If damaged, do not attempt to repair the instrument!
- Earwax in the ear canal may cause a lower temperature reading. Make sure subject's ear canal is clean to ensure an accurate reading. Use caution when cleaning ears.
- The Walgreens Deluxe Instant Ear Thermometer consists of high quality precision parts. Protect it from severe impact and shock. Keep the unit in its storage case when not in use.

WARNING:

- **Please keep the probe covers out of the reach of children. They may present a choking hazard.**
- **Use of this thermometer is not intended as a substitute for consultation with your physician.**
- **This thermometer is not waterproof! NEVER immerse into liquids!**
- **Cleaning the ear canal using improper techniques can harm the inner ear. Follow doctors instructions for proper cleaning.**

3. Product Description

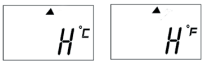
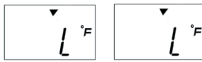
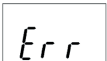
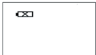
4. How the Walgreen Deluxe Instant Ear Thermometer Measures Body Temperature

The Walgreen Deluxe Instant Ear Thermometer measures the heat radiated from the eardrum and the surrounding tissue. This energy is collected through the lens and converted to a temperature value. The measured reading obtained directly from the eardrum (Tympanic Membrane) can ensure the most accurate body temperature. Measurements taken from the surrounding tissue of the ear canal generate lower readings and may result in misdiagnosis of a fever.

To avoid an inaccurate measurement:


- Use Walgreens probe covers. The unit is calibrated to work with probe covers.
- Straighten the ear canal by gently pulling the middle of the ear back and up. This ensures measurement of heat from the eardrum.
(Please note: Pull outer ear out for newborns - age 1)
- Place the probe firmly into the ear canal. Press the Start button and keep the probe in the ear until the thermometer beeps to indicate completion of the measurement.

The Walgreens Deluxe Instant Ear Thermometer has been clinically tested and proven to be safe and accurate when used in accordance with operating instructions.

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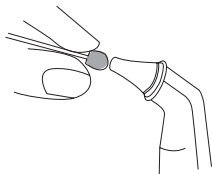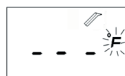
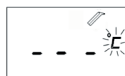
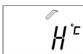
5. How to Read the Display

Display	Display Meaning	Description
 Seen when you turn on the unit Shown for 2 seconds.	Self-check ensures proper functioning.	Press the ON/OFF button to turn on the unit. All icons will be shown for 2 seconds.
	Last reading stored in memory.	The last reading will be shown on the display automatically for 3 seconds.
	Ready to measure.	The unit beeps once and the °C or °F icon is flashing.
	Measurement complete °C or °F stops flashing.	Unit beeps once. When the °C or °F starts to flash you can take another temperature.
	Low battery indication. Unit will not take accurate measurements.	The battery icon will flash to remind the user to replace the battery.

6. How to Apply a New Probe Cover

- (1) Place a probe cover onto the storage case hole with paper side upwards. Align the cover to fit the hole exactly.

- (2) Take the unit, **vertically** penetrate the probe into the **center** part of the probe cover.

- (3) **Completely** push the probe into the probe cover holder hole. Push firmly.

- (4) After feeling a slight click, take out the probe. The cover should be securely attached.

Please Note: Unit is calibrated to work with probe covers. Only use of Walgreens probe covers ensures accuracy and hygienic use.

NOTE:

- In order to avoid spreading germs, please use a new probe cover for each measurement.
- Check to ensure probe cover is fitted on firmly before use (**please see the diagrams below**). If the probe cover is broken, discard and apply a new one immediately.

(X) Incorrect

(✓) Correct

7. Directions for Use

Important: Prior to every measurement, fit a new undamaged Probe Cover on the measuring probe. Failure to do so may result in incorrect temperature measurement.

1. Press the ON/OFF button. The display is activated to show all symbols for 2 seconds.

2. The last measurement reading will be shown on the display automatically for 3 seconds with the "M" symbol visible.

3. A beep sound is heard. The °C or °F symbol will flash. The thermometer is ready to measure.

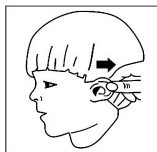
4. Straighten the ear canal by pulling the outer ear up and back to give a clear view of the eardrum.

- For children under 1 year:
- Pull the ear straight back
- Children ages 1 year to adult:
- Pull the ear up and back.

(Caution: Many pediatricians suggest use of ear thermometers for children older than 6 months.)

1 year-adult

Newborn-1 year

5. While pulling the outer the ear, insert the probe snugly into the ear canal and press the "START" button. Release it, when you hear a beep sound. A beep indicates measurement is complete

6. Remove the thermometer from the ear. The display shows the measured temperature.

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7. Replace the probe cover after each measurement. To do this, please follow the instruction in section 6, (How to apply a new probe cover)

8. In order to assure accurate readings, please wait at least 30 seconds after 3-5 continuous measurements.

NOTE:

- For an infant, it is best to have the child laying flat with head sideways so the ear is facing towards you. For an older child or adult, it is best to stand behind and slightly to the side of the patient.
- Always take measurements in the same ear since temperature readings may be different from the right and left ear.
- Wait a few minutes before taking ear temperature after sleeping.
- It is recommended that three temperatures in the same ear be taken and the highest one taken as the reading. This is very important with:
 - 1) Newborn infants in the first 100 days.
 - 2) Children under three years of age with a compromised immune system and for whom the presence or absence of fever is critical.
 - 3) When the user is learning how to use the IR thermometer for the first time until he/she has familiarized himself/herself with the instrument and obtains consistent readings.

Caution:

If you are not confident with readings obtained, confirm temperature with a standard digital thermometer.

8. Changing from Fahrenheit to Celsius and vice-versa

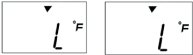
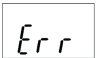
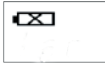
The Walgreens Deluxe Instant Thermometer can display temperature measurements in either Fahrenheit or Celsius. To switch the display between °C and °F, turn the unit off. Press and hold the Start button for 5 seconds, the display will show ---°C or ---°F. Release the Start button. Push the Start button again to toggle between °C or °F. Five seconds after completing the change to °C or °F, the unit will prepare for a measurement, and beep when ready.

9. Error Messages

Display	Meaning	Possible cause and remedy
	Measured Temperature very high	Measured temperature higher than 108.0 °F or 42.2 °C. *Caution!
	Measured temperature low	Measured temperature lower than 32 °C or 89.6 °F. See section 7 for proper technique.


*** Caution: High fever requires immediate medical attention!**

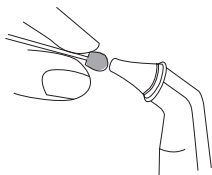Display / Problem	Display Meaning	Possible cause and fault remedy
	Room temperature too high	Displays "H" in conjunction with the "▲" when room temperature is higher than 40.0 °C or 104.0 °F.
	Room temperature too low	Display "L" in conjunction with the "▼" when room temperature is lower than 10.0 °C or 50.0 °F.
	System malfunction	Check battery. Possible damage from dropping or water.
	Blank display	Dead battery? Check battery polarity. (+ and -)
	Low battery indication	Battery should be replaced immediately.

10. Cleaning and Storage

Use an alcohol swab or cotton swab moistened with alcohol (70% Isopropyl) to clean the thermometer casing and the measuring probe. Ensure that no liquid enters the interior of the thermometer. **Never use abrasive cleaning agents, thinners or gasoline for cleaning and never immerse the instrument in water or other cleaning liquids.** Take care not to scratch the surface of the lens or display. Remove the battery from the instrument if it is not required for extended periods of time in order to avoid damage to the thermometer resulting from a leaking battery.

Please Note: Ear wax or dirt on the lens will result in an inaccurate reading. Clean the lens as indicated above.

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
13. Technical Specifications

Type:	Instant Ear Thermometer 666188
Measuring Range:	32.0 - 42.2 °C or 89.6 - 108.0 °F
Accuracy:	Laboratory: : ±0.2 °C, 32.0 ~ 42.2 °C ±0.4 °F, 89.6 ~ 108.0 °F
Display:	Liquid Cristal Display with 0.1 °C (0.1 °F) resolution.
Beeps When:	a. The unit is turned ON and ready for measurement: (1 short “beep” sound) b. Measurement complete: 1 long beep sound. c. System error, malfunction, or “Hi”/“Low” measurement. 3 short “beep” sounds.
Memory:	Automatically displays the last measured temperature.
Illuminated Display: (Back-Lite)	a. The display will be lighted for 4 seconds when the unit is turned ON. b. The display will be lighted again for 5 seconds when the measurement has been completed.
Operating temperature:	10°C to 40 °C (50.0 °F to 104 °F)
Storage/transport temperature:	-25 °C to +55 °C (-13 °F to 131 °F)
Automatic Switch-off:	Approx. 1 minute after last measurement has been taken.
Battery:	1 each CR2032 BATTERY - at least 1000 measurements
Dimensions:	141mm (L) x 26 mm (W) x 20 mm (H)
Weight:	45g (with battery), 40g (w/o battery)
Standards:	Manufactured to meet European PrEN12470-5 and U.S. ASTM E-1965 requirements

Please observe the applicable disposal regulations.

11. Replacing the Battery

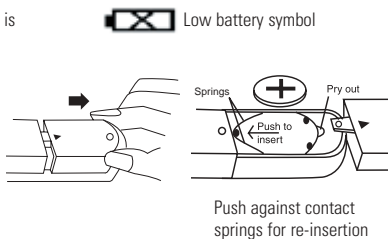The Walgreens Deluxe Instant Ear Thermometer is supplied with one lithium battery, type CR2032. Replace with a new CR2032 battery when the low battery symbol appears on the display. Use a screwdriver to loosen the screw from battery cover. Remove the battery cover and replace battery. Do not overtighten screw when replacing the cover.

12. Warranty

This product was manufactured with the utmost care according to international quality standards. This product is warranted for 2 years against manufacturing defects. Damage resulting from incorrect use or abuse is not covered by the warranty. Battery and packaging are excluded from the warranty. Claims beyond this, including claims for damages, are excluded.

Direct product complaints to:
Microlife USA, Inc.,
424 Skinner Blvd. Suite C,
Dunedin, FL 34698.

custserv@microlifeusa.com.
1-888-314-2599.
www.microlife.com
Or return to your Walgreens store.

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