microlife®

INSTRUCTION MANUAL

Bluetooth®
Blood
Pressure
Monitor

Model # BP3GY1-2N

QUESTIONS? 1-800-568-4147

Important Product and Safety Information

Follow Instructions for Use. This document provides important product operation and safety information regarding this Blood Pressure Monitor. Please read this document thoroughly before using the device and keep for future reference.

This blood pressure monitor is an automatic digital blood pressure measuring device for use by adults, including those with Diabetes mellitus, on the upper arm at home or in the doctor's office. It enables a very fast and reliable measurement of the systolic and diastolic blood pressure, as well as the pulse rate, by way of the oscillometric method. This device detects the appearance of irregular heartbeats during measurement and provides a warning signal when the irregular heartbeat is detected. The device can be connected via Bluetooth® to a smart mobile device running the Microlife Connected Health App.

This blood pressure monitor IS intended to be used:

- For self-measurement/monitoring of blood pressure and pulse in adults, including those with Diabetes mellitus; whereas the person being measured may be the user/operator of the device.
- Within a home healthcare environment.
- With a cuff located upon the user's upper arm; ½ inch above the elbow over the artery as indicated on the cuff.

This blood pressure monitor IS NOT intended for use with:

- Pregnancy
- Preeclampsia
- Children under 12
- Neonatal patients

WARNING – Potentially hazardous situation that if not avoided may result in serious injury or death.

- Self-measuring means monitoring, not diagnosis or treatment. Unusual values must always be discussed with your doctor. Under no circumstances should you independently alter the dosages of any drugs prescribed by your physician.
- Consult your physician before using this device if any of the following or similar conditions are present: arrhythmias such as atrial or ventricular premature beats or atrial fibrillation, arterial sclerosis, poor

perfusion, diabetes, age, pregnancy, preeclampsia, renal diseases. Motion during measurement, including trembling or shivering may affect the measurement.

- Though not for use with children under 12, ensure that any children around this device are supervised; some parts are small enough to be swallowed and any present tubes or cables may provide a risk of strangulation.
- Ensure the cuff tubing is not kinked during use as harmful injury may occur due to the effect of blood flow interference caused by high pressure in the cuff not releasing.

DO NOT:

- × Use this device if you think it is damaged or anything appears unusual (e.g. sporadic operation, open packaging upon purchase).
- × Use the displayed pulse for checking the frequency of heart pacemakers as this device is not suitable for this action.
- × Open/modify this device; inaccuracy and/or harmful injury may result.
- × Conduct many frequent measurements as this may result in harmful injury due to blood flow interference.
- × Place the Cuff over a wound as this may cause further injury.
- × Place and pressurize the Cuff over/near any present intravascular access or therapy, or arteriovenous shunt, as this may cause blood flow interference and result in harmful injury.
- × Place and pressurize the Cuff over a limb near the side of a mastectomy as this may cause harmful injury.
- × Use the Blood Pressure Monitor on a limb simultaneously with other medical equipment on the same limb due to possible interference with such medical equipment.
- × Maintain pressure in the Cuff applied to the limb for a prolonged amount of time. Ensure that circulation in the limb is not impaired by checking circulation if prolonged/repeated exposure to pressure occurs.
- × Service the device when being used or when power is supplied. When the device is not in use and power is removed, cuff assembly, batteries and ac adapter (if applicable) may be replaced by the user with Microlife supplied replacements. No other parts/components are accessible.

CAUTION – Potentially hazardous situation that if not avoided may result in minor/moderate injury, property damage, and/or damage to the device

- This device contains sensitive electronics components. Avoid strong electrical or electromagnetic fields in the direct vicinity of the device (e.g., mobile telephones, microwave ovens). These can lead to temporary impairment of the measuring accuracy. Move the device to another location if interference is determined.
- When not using the Blood Pressure Monitor for extended lengths of time, remove the batteries to avoid potential battery leakage and damage to the monitor.
- When replacing the batteries, ensure all the batteries are replaced simultaneously to avoid battery damage and potential damage to the monitor. Microlife USA does not recommend using rechargeable batteries.
- To avoid inaccurate measurements and to lessen any discomfort from Cuff pressure, ensure the Cuff is placed correctly on the limb and fits correctly when snug (not tight), as indicated by markings with the Cuff.
- Consult your physician in cases of frequent irregular heartbeat detections.
- This Risk Indicator feature is provided in order to help you understand your potential blood pressure risk. However, this feature is neither a diagnosis nor a substitute for a medical examination. It is important to consult with your physician to determine your risk.

DO NOT:

- × Drop this device or expose it to strong vibrations; sensitive components may be affected resulting in inaccuracies and/or operational issues.
- × Use the Blood Pressure Monitor outside of its specified operation temperature and humidity rating, or if stored outside of its specified storage temperature and humidity rating. Avoid storage in direct sunlight.
- × Use this device in a moving vehicle; inaccurate measurements may result.
- × Use third party accessories. Only use Microlife authorized accessories, such as cuffs or AC adapters, as those not approved for use with the device may provide inaccurate measurements, injury, and/or damage the device.

STANDARDS

In addition to the standards stated in the Instruction Manual:

- This medical device is compliant with medical device and non-invasive blood pressure monitor standards IEC 60601-1, IEC 60601-1-2, IEC 60601-1-11, and AAMI/ANSI/IEC 80601-2-30, and
- Electromagnetic standards IEC 60601-1-2 along with FCC Part 15, and
- Clinical Testing per standard ISO 81060-2:2013 was conducted on blood pressure device using the same measurement technology.

Please note: According to international standards, your monitor should be checked for accuracy every 2 years.

TYPE

IP20 Protected against solid foreign particles with a diameter of more than 12.5 mm, no protection against water. Keep Dry. 

Type BF Applied Part

Batteries and electronic instruments must be disposed of in accordance with the locally applicable regulations, not with domestic waste.

Expected Life

Monitor 5 Years

Cuff 2 Years

BHS

The B.H.S. (British Hypertension Society) clinical protocol was used to measure the accuracy of this product. Blood pressure units using the same measurement technology are graded "AA" for systolic/diastolic accuracy by independent investigators using the BHS protocol. This is the highest grading available for blood pressure monitors. Please see bhsoc.org for more information. (Uses the same algorithm as B.H.S. graded "AA" model number 3BT0-1)

FCC

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation. Changes or modifications to the product are not approved by Microlife USA and could void the user's authority to operate the equipment under FCC jurisdiction.

This equipment has been verified to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment on and off, the user is encouraged to try to correct the interference by one or more of the following measures: 1) Reorient or relocate the receiving antenna, 2) Increase the separation between the equipment and receiver, 3) Connect the equipment into an outlet on a circuit different from that to which the receiver is connected. And 4) Consult the dealer or an experienced radio/TV technician for help.

This device complies with FCC radiation exposure limits set forth for an uncontrolled environment. End users must follow the specific operating instructions for satisfying RF exposure compliance.

Trademark Usage:

Apple, the Apple logo, iPad, and iPhone are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc.

Android and Google Play are both trademarks of Google Inc.

The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks in this Blood Pressure Monitor is under license. Other trademarks and trade names are those of their respective owners.

Bluetooth® Automatic Blood Pressure Monitor

Instruction Manual

Table of Contents

1. Introduction

- 1.1. Your automatic blood pressure monitor
- 1.2. Diabetes mellitus validation

2. Components of your blood pressure monitor

- 2.1. Blood pressure monitor
- 2.2. Wide Range Cuff cuff

3. How do I get started?

- 3.1. Protective screen film
- 3.2. Inserting the batteries
- 3.3. Power switch / screen lock
- 3.4. Using the interactive touch screen
- 3.5. Setting the date and time
- 3.6. Select the user
- 3.7. Select the measuring mode: standard or Microlife Averaging Mode
- 3.8. Microlife Averaging Mode (MAM)
- 3.9. Connecting the cuff and monitor
- 3.10. Using the AC power adapter

4. How do I take a measurement?

- 4.1. Preparing to take a measurement
- 4.2. Common errors
- 4.3. Using the cuff
- 4.4. Taking a measurement
- 4.5. Memory: reviewing readings
- 4.6. Stopping a measurement
- 4.7. Hypertension risk indicator
- 4.8. Irregular heartbeat detector
- 4.9. Battery change indicator

5. Bluetooth® functions & application (app)

- 5.1. Activating your Bluetooth®
- 5.2. Pairing your device
- 5.3. Unpairing your device

6. Error messages / troubleshooting

7. How is blood pressure measured?

7.1. What is blood pressure, and why does it fluctuate?

7.2. Measurement guidelines

7.3. What can I do to change my blood pressure?

8. Care and maintenance

9. Limited warranty

10. Technical specifications

11. How to contact us

1. Introduction

1.1. Your automatic blood pressure monitor

Thank you for purchasing a fully automatic blood pressure monitor. Your monitor is designed to provide fast and reliable digital readings of your pulse, and systolic / diastolic blood pressure using the oscillometric method on your upper arm. It offers clinically proven accuracy and has been designed to be user friendly.

If you have additional questions regarding blood pressure measurements please contact Microlife Customer Service at 1-800-568-4147 or contact your doctor.

1.2. Diabetes mellitus validation

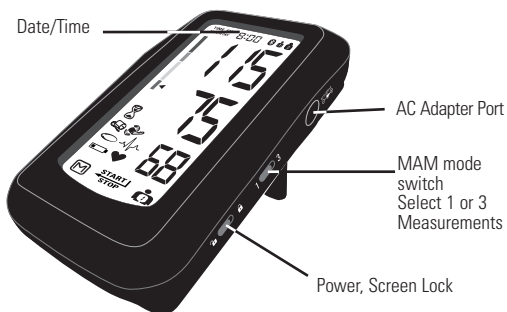
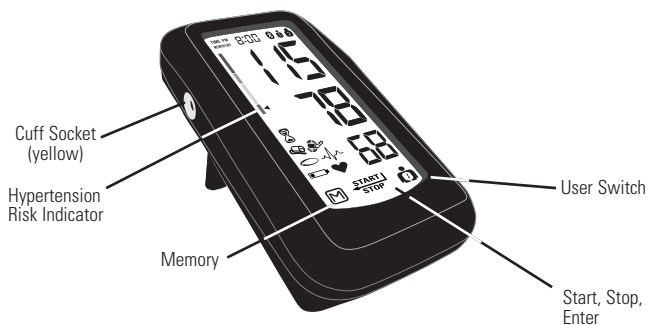
Diabetes, and stiff arteries, can make accurate blood pressure readings a challenge. Microlife's commitment to health has led to a breakthrough clinical validation for accurate readings in the presence of diabetes. This innovation reinforces our commitment to quality, accuracy, and helping you take control of your health.

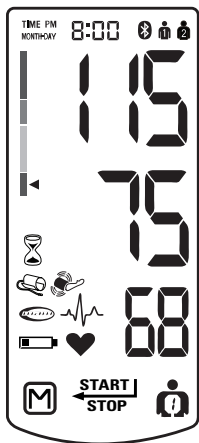
Diabetes mellitus (or diabetes) is a chronic, lifelong condition that affects your body's ability to use the energy found in food. There are three major types of diabetes: type 1 diabetes, type 2 diabetes, and gestational diabetes.

Note: this monitor does not measure blood glucose.

2. Components of your blood pressure monitor

2.1. Blood pressure monitor

— Systolic

— Diastolic

— Pulse

-  — User 1 & 2
-  — Guest
-  — Bluetooth®
-  — Battery Indicator
-  — Forward
-  — Backward
-  — Memory
-  — Enter
-  — MAM
-  — Irregular Heartbeat
-  — Pause
-  — Heartbeat
-  — Error 2
-  — Error 3

2.2. Wide Range Cuff:

For arm circumference 8.7" - 16.5" (22 cm - 42 cm)

If you ever need to buy a replacement cuff, call us toll-free at 1-800-568-4147.

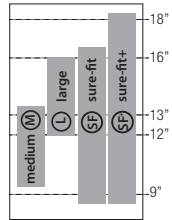
Anti-Slip Cuff:

Your wide range cuff is equipped with anti-slip grips that ease the process of correctly positioning the cuff on your arm.

Cuff Sizing:

The proper fit is essential for accurate blood pressure readings. The average arm circumference for a woman is 11", and the average for a man is 13". This cuff can accommodate arm circumferences from 8.7" to 16.5" and fits most adults.

Arm circumference should be measured with a measuring tape in the middle of the relaxed upper arm. If the cuff is too small, call 1-800-568-4147 for additional information.

Note:

- Do not force cuff connection into the port opening.
- Make sure the cuff connection is not pushed into the AC adapter port.

3. How do I get started?

3.1. Protective screen film

There is a protective piece of static film covering the monitor's screen. Please remove it before use.

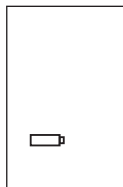
3.2. Inserting the batteries

After you have unpacked your device, insert the batteries. The battery compartment is located on the back of the device.

- a) Remove the battery cover.
- b) Insert the batteries (4 x size AAA 1.5 V), observing the indicated polarity.
- c) If a battery warning appears in the display, the batteries are discharged and must be replaced.

Attention!

- After the low battery indicator appears, the device won't function until the batteries have been replaced.
- Please use AAA 1.5 V batteries and replace them all at the same time.
- If the blood pressure monitor is not used for long periods, remove the batteries from the device.
- We do not recommend using rechargeable batteries.

Low Battery Indicator

3.3. Power switch / screen lock

To activate your monitor, push the switch on the right hand side of the monitor to the unlocked position.

unlocked

locked

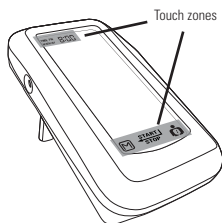
3.4. Using the interactive touch screen

This monitor features an interactive screen with 2 touch zones which operate similarly to traditional buttons, but require only a light touch of the finger to operate.




The following functions are made easier through the interactive touch screen:

- Setting time and date
- Scrolling through memories
- Selecting the user
- Starting/stopping the blood pressure measurement

3.5. Setting the date and time

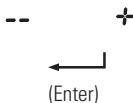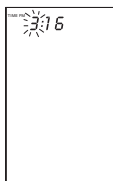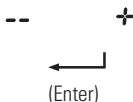1. After the new batteries are inserted and the monitor is unlocked, the year number flashes in the display. You can advance the year by pressing the plus or minus icon. To confirm and then set the month, press enter.
2. You can now set the month using the minus and plus. To confirm and then set the day, press enter.
3. Please follow the instructions above to set the day, hour and minutes.
4. Once you have set the last minute and pressed enter, the date and time are set and the time is displayed.
5. If you want to change the date and time, press and hold the time zone down for approximately 3 seconds until the year number starts to flash. Now you can enter the new values as described above.
6. To bypass setting date and time, press the time area of the touch screen.

3.6. Select the user

This blood pressure monitor is designed to store 120 measurements for each of two users. Memories are not stored in guest mode.

Guest Mode

Before taking a measurement, be certain that the correct user has been selected.

- With the monitor unlocked, press and hold the user icon, then release.
- The user icon will begin to blink. Tap again to change users.
- To confirm your selection wait 3 seconds, or press STOP.

3.7. Select measuring mode: standard or Microlife Averaging Mode (MAM)

This instrument enables you to select either standard (single measurement) or measurement averaging mode (automatic triple measurement).

- To select Standard mode, push the switch on the side of the monitor to Position "1."
- To select Averaging mode, push the switch to Position "3." Note the "MAM" icon will illuminate on the screen.

If you select 1, then only one measurement will be taken. If you select 3, the unit will inflate and deflate three times resulting in one final averaged measurement.

3.8. Microlife Averaging Mode (MAM)

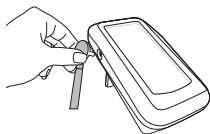
- In Microlife Averaging Mode (MAM), three measurements are automatically taken in succession and the result is then automatically analyzed and displayed. Because your blood pressure constantly fluctuates, a result determined in this way is more reliable than one produced by a single measurement.
- After pressing the START button the selected Microlife Averaging Mode appears in the display as the MAM symbol.
- The bottom, right-hand section of the display shows a 1, 2 or 3 to indicate which of the 3 measurements is currently being taken.

3.9. Connecting the cuff and monitor

Insert the yellow cuff tube connector into the yellow socket on the left side of the instrument.


If your cuff connector is another color, please contact customer service as additional steps are needed to obtain an accurate reading.

3.10. Using the AC power adapter

You may also operate this monitor using the included AC adapter. Use only the included AC adapter to avoid damaging the unit.

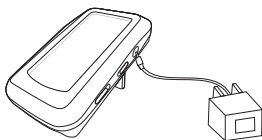- Ensure that the AC adapter and cable are not damaged.
- Plug the adapter cable into the AC adapter port on the right side of monitor.
- Plug the AC adapter into a 110 V power socket (U.S. or Canada).
- Test that power is available by pressing the START button.

Note:

- No power is taken from the batteries while the AC adapter is connected to the instrument.
- If the power is interrupted during a measurement (e.g., by removal of the adapter from the wall socket), the instrument must be reset by removing the plug from the instrument.

If you have any questions regarding the AC adapter, call us at 1-800-568-4147.

4. How do I take a measurement?

Please note: You should always be seated before and during measurement.

4.1. Preparing to take a measurement

- Avoid eating and smoking as well as all forms of exertion directly before measurement. These factors influence the measurement result. Find time to relax by sitting in an armchair in a quiet atmosphere for about 5 minutes before your measurement.
- Remove any garment that fits closely to your upper arm.
- Always measure on the same arm (normally left).
- Compare measurements at the same time of day, since blood pressure changes during the course of the day (as much as 20–40 mm Hg).

4.2. Common errors

Note: Comparable blood pressure measurements always require the same conditions. Conditions should always be quiet.

- All efforts by the user to support the arm can increase blood pressure. Make sure you are in a comfortable, relaxed position and do not flex any of the muscles in the measurement arm during the measurement. Use a cushion for support if necessary.
- If the arm artery lies considerably lower or higher than the heart, an erroneously high or low blood pressure will be measured. Each 15 cm (6") difference in height between your heart and the cuff results in a measurement error of 10 mm Hg.
- Cuffs that are too narrow or too short result in false measurement values. Selecting the correct cuff is extremely important. Cuff size is dependent upon the circumference of the arm (measured in the center). The permissible range is printed on the cuff. If this is not suitable for your use, please call 1-800-568-4147.
- A loose cuff or a sideways protruding air pocket causes false measurement values.
- With repeated measurements, blood accumulates in the arm, which can lead to false results. Consecutive blood pressure measurements should be repeated after a 1 minute pause or after your arm has been held up in order to allow the accumulated blood to flow away.

4.3. Using the cuff

a) Pass the end of the cuff through the flat metal ring so that a loop is formed. The hook and loop material must be facing outward. (Ignore this step if the cuff has already been prepared.)

b) Slide the cuff onto your upper left arm. The rubber tube should be on the inside of your arm extending downward to your hand. Make certain the cuff lies approximately $\frac{1}{2}$ " (1 to 2 cm) above the elbow.

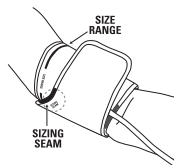
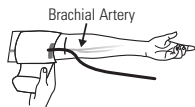
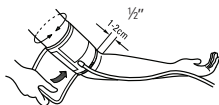
IMPORTANT: The red strip on the edge of the cuff (Artery Mark) must lie over the artery which runs down the inner side of the arm.

TIP: Align red artery mark to pinky finger.

c) To secure the cuff, wrap it around your arm and press the hook and loop material together. Check the position of the green SIZING SEAM on the cuff. Ensure green sizing seam overlaps green size range bar.

d) There should be little free space between the arm and the cuff. You should be able to fit 2 fingers between your arm and the cuff. Clothing must not restrict the arm. Any piece of clothing which does must be removed. Cuffs that don't fit properly result in false measurement values. Measure your arm circumference if you are not sure of proper fit.

8.7" - 16.5"


- e) Lay your arm on a table so the cuff is at the same height as your heart. Make sure your arm is relaxed and the tube is not kinked.
- f) Remain seated quietly for 5 minutes before you begin the measurement.

Comment

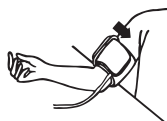If it is not possible to fit the cuff to your left arm, it can also be placed on your right arm. However, all measurements should be made using the same arm. Comparable blood pressure measurements always require the same conditions (relax for several minutes before a reading).

Cuff on
right arm

4.4. Taking a measurement

After the cuff has been appropriately positioned the measurement can begin. Remain still during your measurement, do not flex muscles, and refrain from talking.

- Remain seated quietly for 5 minutes before you begin the measurement.
- Press the START button. After the system check, the monitor displays OK and the pump begins to inflate the cuff. On the display, the increasing cuff pressure is continually shown.
- After automatically reaching an individual pressure, the pump stops and the pressure will deflate slowly. The cuff pressure is displayed during the measurement.
- When the device has detected your pulse, the heart symbol in the display begins to blink.
- When the measurement has been concluded, the air will automatically release from the cuff. The measured systolic and diastolic blood pressure values, as well as the pulse, are now displayed.
- The measurement results are displayed until you switch the device off. If no button is pressed for 1 minute, the device switches off automatically.
- When the unit is set to the MAM (Microlife Averaging Mode) setting, 3 separate measurements will take place in succession, after which your result is calculated and displayed as a single, averaged measurement (individual results are not displayed). There is a 15 second resting time between each measurement. A count-down indicates the remaining time between measurements.

If one of the measurements causes an error message, it will be repeated one more time. If any additional error occurs, the measurement will be discontinued and error code displayed.

Pumping Pressure

Measuring

Systolic

Diastolic

Pulse

Measurement Complete

4.5. Memory: reviewing readings

At the end of a measurement, this monitor automatically stores each result with date and time. This unit stores 120 memories for each of 2 users.

Viewing the stored values

- With the unit unlocked, press "M." The display first shows "A," then shows an average of all measurements stored in the unit for that user.
- Press the plus or minus icon to scroll between stored readings.

Please note: Measurements for each user are averaged and stored separately. Be certain that you are viewing the measurements for the correct user.

- To exit, press STOP.

Memory full

When the memory has stored 120 results, a new, measured value is stored by overwriting the oldest value.

Clear all values

- If you are sure that you want to permanently remove all stored values, hold down "M" until "CL" appears and then release "M".
- To continue and erase the values, press "M" again while "CL" is flashing.
- To discontinue with the clearing of memories, press "STOP."
- Individual values cannot be cleared.

4.6. Stopping a measurement

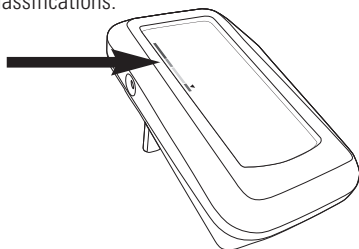
If it is necessary to interrupt a blood pressure measurement for any reason (e.g., the patient feels unwell), the "STOP" button can be pressed at any time. The device then immediately lowers the cuff pressure automatically and enters sleep mode.

4.7. Hypertension risk indicator

The bars on the left-hand edge of the display show the possible range for a blood pressure reading. The reading can fall within the normal (green), borderline (yellow) or danger (orange, dark orange) range.

The classification is based on standards established by the American Heart Association (AHA) and American College of Cardiology (ACC) in 2017.

Refer to the chart in section 7.2 of this instruction manual for details of the classifications.

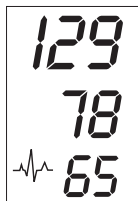
A triangle is displayed to the right of the traffic light bar according to your measurement. If the display shows a triangle in the:

- **green zone**, your measurement is "Normal."
- **lower yellow zone**, it is "Elevated."
- **upper yellow zone**, it is "Stage 1 Hypertensive."
- **orange zone**, it is "Stage 2 Hypertensive."
- **lower dark orange zone**, it is "Stage 2 Hypertensive."
- **upper dark orange zone**, it is "Hypertensive Crisis."

Dk. Orange	Dk. Orange	Dk. Orange	Dk. Orange	Dk. Orange	Dk. Orange
Orange	Orange	Orange	Orange	Orange	Orange
Yellow	Yellow	Yellow	Yellow	Yellow	Yellow
Green	Green	Green	Green	Green	Green
"Normal" Blood Pressure Reading	"Elevated" Blood Pressure Reading	"Stage 1 Hypertensive" Blood Pressure Reading	"Stage 2 Hypertensive" Blood Pressure Reading	"Hypertensive Crisis" Blood Pressure Reading	

4.8. Irregular heartbeat detector

The appearance of this symbol  indicates that certain pulse irregularities were detected during the measurement. In this case, the result may deviate from your normal basal blood pressure – repeat the measurement. In most cases, this is no cause for concern. However, if the symbol appears on a regular basis (e.g., several times a week with measurements taken daily), we advise you to tell your doctor.

Please show your doctor the following explanation:

Information on frequent appearance of the irregular heartbeat symbol

This instrument is an oscillometric blood pressure monitor device that also analyzes pulse frequency during measurement. The instrument is clinically tested.

If pulse irregularities occur during the measurement, the irregular heartbeat symbol is displayed with the measurement.

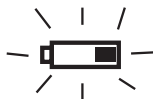
If the symbol appears frequently or if it suddenly appears more often than usual, we recommend the patient seek medical advice.

The instrument does not replace a cardiac examination but serves to detect pulse irregularities at an early stage.

4.9. Battery change indicator

Batteries almost discharged

When the batteries are approximately 75% used, the battery symbol will flash a few times as soon as the instrument is switched on. Although the instrument will continue to measure reliably, you should obtain replacement batteries.

Batteries discharged – replacements required

When the batteries are discharged, the battery symbol will appear, unblinking, as soon as the instrument is switched on. You cannot take any additional measurements and must replace the batteries.

1. Open the battery compartment on the bottom of the instrument.
2. Replace all of the batteries at the same time, ensure they are correctly connected, as shown on the symbols in the compartment.
3. The memory retains all values although date and time (and possibly also set alarm times) must be reset – the year number will flash automatically after the batteries are replaced.
4. To set date and time, follow the procedure described in section 3.2.

Note:

Use four new AAA 1.5 V batteries. Do not use batteries beyond their expiration date. If the monitor is not going to be used for a prolonged period, the batteries should be removed.

We do not recommend using rechargeable batteries.

5. Bluetooth® functions and application (App)

Please download the Microlife Connected Health App

(referred to as App in this document) from Apple's App Store® or Google Play® before pairing your devices.:

Questions?

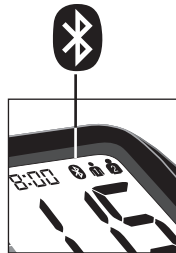
- To access the tutorial, go to the App home screen (Microlife Connected Health) and press the "Settings & Help" button.
- Press "Tutorial" and select your topic.
- If you have any issues with your setup, please contact customer service toll free: 1-800-568-4147 for additional support.

5.1 Activating your Bluetooth®

The bluetooth® icon indicator on your blood pressure monitor, located in the upper left portion of the screen, is designed to provide information about the connection between your device and the monitor. Please note the following:

- The Bluetooth® icon on your blood pressure monitor will flash initially when ready to connect with a device, and continue flashing for 2 minutes until a connection occurs.
- To manually activate the Bluetooth® on your monitor, press and hold the START button for approximately 3 seconds or until the Bluetooth® icon starts blinking.
- If a connection does not occur within 2 minutes, the Bluetooth® function on the monitor will automatically turn off.
- Note: At least one blood pressure reading must be taken and recorded on the monitor before the data can be transferred

Bluetooth® icon

5.2. Pairing your device:

- Download the App to your phone.
- Create User Profile on App.
- Confirm phone's Bluetooth® is turned on.
- Open the App on your phone.
- Select "Dashboard" from the App's home screen.
- Confirm at least one blood pressure reading has been taken and recorded on the monitor. This needs to occur before the data can be transferred.
- Press and hold the START button on the monitor for approx. 3 seconds or until the Bluetooth® icon starts blinking.
- On the App press the "sync" button located on the "Dashboard" screen.
- Data in monitor should successfully transfer over to the App on your phone.

Sync

Setting up your blood pressure monitor:

Confirm the correct user is selected on your blood pressure monitor:

- Confirm the user icon on the top right hand side of the monitor screen is set to user 1 or 2.
- To change users on your monitor, see section 3.6.

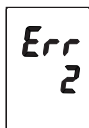
5.3. Unpairing a device

- Delete the monitor pairing from your phone.
- Clear the phone pairing on blood pressure monitor.
 - Press and hold the "START" button.
 - The Bluetooth® icon will turn on and start blinking.
 - "Pr" will appear in the bottom right hand corner of the screen.
 - "CL" will appear, and then disappear in the top right corner.
- Once the letters disappear, the monitor has been successfully cleared of all pairings. You can remove your finger from the "START" button.

6. Error messages / troubleshooting

If an error occurs during a measurement, the measurement is discontinued and a corresponding error code is displayed (example: Error no. 2).

Error No. Possible cause(s)/Solutions

ERR 1 Weak signal. The tube may have loosened, or no pulse was detected.* Ensure cuff connections are tight with proper cuff placement. See section 4.3.

ERR 2  Movement detected. Repeat measurement, keeping still and quiet.

ERR 3  No pressure detected. Inflation of the cuff takes too long. The cuff is not correctly seated or the hose connection is not tight. Re-position cuff and repeat the measurement.

ERR 5 Abnormal reading. The measured readings indicate an unacceptable difference between systolic and diastolic pressures. Take another reading following directions carefully. Contact your doctor if you continue to get unusual readings.

HI The cuff pressure is too high. Relax for 5 minutes and repeat the measurement.*

LO The pulse is too low (less than 40). Repeat the measurement.*

***If this or any other problem occurs repeatedly, please consult your doctor.**

Other possible errors and their solutions

If problems occur when using the device, the following points should be checked and, if necessary, the corresponding measures are to be taken:

Malfunction	Remedy
The display remains blank when the instrument is switched on although the batteries are in place.	<ol style="list-style-type: none">1. Check batteries for the correct polarity.2. If the display is unusual, remove the batteries and exchange them for new ones.
The pressure does not rise although the pump is running.	Check the connection of the cuff tube and connect properly.
The device frequently fails to measure blood pressure values or the values measured are too low or high.	<ol style="list-style-type: none">1. Check the positioning of the cuff.2. Measure blood pressure again in peace and quiet, carefully following the details in section 4.
Every measurement results in a different value, although the device functions normally and normal values are displayed.	Please read the following information and points listed in section 4.2 " Common errors. " Repeat the measurement. Please note: Blood pressure fluctuates continually so successive measurements will show some variability.
Blood pressure values differ from those measured by my doctor.	Record the daily development of the measured values and consult your doctor (see section 4.1) Please note: Individuals visiting their doctor frequently experience anxiety which can result in a higher reading than at home under resting conditions.
After the instrument has inflated the cuff the pressure falls very slowly, or not at all. (No reasonable measurement possible.)	<ol style="list-style-type: none">1. Check cuff connections.2. Ensure the unit has not been tampered with.

7. How is blood pressure measured?

7.1. What is blood pressure, and why does it fluctuate?

Your level of blood pressure is determined in the circulatory center of the brain and adjusts to a variety of situations through feedback from the nervous system. To adjust blood pressure, the strength and frequency of the heart (pulse), as well as the width of circulatory blood vessels is altered. Blood vessel width is affected by fine muscles in the blood vessel walls.

Your level of arterial blood pressure changes periodically during heart activity. During the "blood ejection" (Systole), the value is highest (systolic blood pressure value). At the end of the heart's "rest period" (Diastole), pressure is lowest (diastolic blood pressure value).

Blood pressure values must lie within certain normal ranges in order to prevent particular diseases.

Blood pressure is very high if your systolic blood pressure is over 140 mmHg and/or your diastolic pressure is above 90 mmHg, **while at rest**. In this case, please consult your doctor immediately. Long-term values at this level endanger your health due to continual damage to the blood vessels in your body.

Should the systolic blood pressure values lie between 130 mmHg and 139 mmHg and/or the diastolic blood pressure values lie between 80 mmHg and 89 mmHg, consult your doctor. Regular self-checks will be necessary.

If you have blood pressure values that are too low (i.e., systolic values under 105 mm Hg and/or diastolic values under 60 mm Hg), consult your physician.

Even with normal blood pressure values, a regular self-check with your blood pressure monitor is recommended. You can detect possible changes in your values early and react appropriately.

If you are undergoing medical treatment to control your blood pressure, keep a record of values along with time of day and date. Show these values to your physician.

Never use the results of your measurements to independently alter the medication prescribed by your physician.

7.2. Measurement guidelines

The following standards for assessing high blood pressure (in adults) have been established by the American Heart Association (AHA) and American College of Cardiology (ACC) in 2017.

Category	Systolic (mm Hg)	Diastolic (mm Hg)
Normal	<120	and <80
Elevated	120-129	and <80
Hypertension		
Stage 1 Hypertension	130-139	or 80-89
Stage 2 Hypertension	140-179	or 90-119
Hypertensive Crisis	≥180	or ≥120

Additional information

- This chart reflects a 2017 update to blood pressure standards.
- If your values are mostly normal under resting conditions but exceptionally high under conditions of physical or psychological stress, it is possible that you are suffering from so-called "labile hypertension." Consult your doctor.
- Correctly measured diastolic blood pressure values above 120 mm Hg require immediate medical treatment.

7.3. What can I do to change my blood pressure?

- a) Consult your doctor.
- b) Increased blood pressure values (various forms of hypertension) are associated with considerable health risks over time. Arterial blood vessels in your body are endangered due to constriction caused by deposits in the vessel walls (arteriosclerosis). A deficient supply of blood to important organs (heart, brain, muscles) can result from arteriosclerosis. Furthermore, the heart will become structurally damaged with increased blood pressure values.
- c) There are many different causes of high blood pressure. We differentiate between the common primary (essential) hypertension and secondary

hypertension. The latter group can be ascribed to specific organ malfunctions. Please consult your doctor for information about the possible origins of your own increased blood pressure values.

d) There are measures which you can take to reduce and even prevent high blood pressure. These measures must be permanent lifestyle changes.

1) Eating habits

- Strive for a normal weight corresponding to your age. See your doctor for your ideal weight.
- Avoid excessive consumption of common salt.
- Avoid fatty foods.

2) Previous illnesses

- Consistently follow all medical instructions for treating illness such as:
 - Diabetes (diabetes mellitus)
 - Fat metabolism disorder
 - Gout

3) Habits

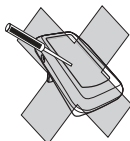
- Give up smoking completely.
- Drink only moderate amounts of alcohol.
- Restrict your caffeine consumption (e.g., coffee).

4) Physical constitution

- **After a preliminary medical examination, do regular exercise.**
- Choose sports which require stamina and avoid those which require strength.
- Avoid reaching the limit of your performance.
- With previous illnesses and/or an age of over 40 years, please consult your doctor before beginning your exercise routine. You must receive advice regarding the type and extent of exercise that is appropriate for you.

8. Care and maintenance

- a) The cuff contains a sensitive airtight bubble. Handle this cuff carefully and avoid all types of stress through twisting or buckling.
- b) Clean the device with a soft, dry cloth. Do not use gasoline, thinners or similar solvents. Spots on the cuff can be removed carefully with a damp cloth and soapsuds. **Do not wash the cuff in a dishwasher, clothes washer or submerge it in water.**
- c) Handle the tube carefully. Do not pull on it. Do not allow the tubing to kink and keep it away from sharp edges.
- d) **Never open the monitor.** This voids the manufacturer's warranty.

9. Limited warranty

Your Automatic Blood Pressure Monitor is **warrantied for 5 years** by Microlife USA Inc, against manufacturer defects for the original purchaser only, from date of purchase.

The 5 year warranty applies to the monitor only. The following accessories are warrantied for 1 year: cuff and AC adapter.

Batteries are not covered by this warranty.

The warranty does not apply to consequential and incidental damages, or damage caused by batteries, improper handling, and accidents. Professional use, not following the operating instructions, and alterations made to the monitor or accessory by third parties, are also not included in this warranty. Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitation or exclusion may not apply to you.

Microlife USA Inc will investigate your concern. A monitor or accessory as defined by this warranty, determined to be out of specification, will be replaced and shipped to you at no cost. A monitor or accessory as defined by this warranty, determined to be within specification, will be returned to you with a report of findings, at no cost.

Please use the below customer service contact information to reach Microlife USA Inc. regarding any warranty concerns. We ask that you please contact us before sending any product back in order to better identify, and more quickly process, your concern.

This warranty gives you specific legal rights, and you may also have other rights which vary from state to state.

10. Technical specifications

Weight:	.320 g (with batteries)
Size:	.86 (W) x 148 (L) x 43 (H) mm
Storage temperature:	-.20 to +55°C (-4° to +131°F)
Operation/Storage Humidity:	.15 to 90% relative humidity maximum
Operation temperature:	.10 to 40°C (50° to 104°F)
Display:	.LCD (Liquid Crystal Display)
Measuring method:	.Oscillometric
Pressure sensor:	.Semiconductor
Measuring range:	
SYS:	.60 to 255 mm Hg
DIA:	.40 to 200 mm Hg
Pulse:	.40 to 199 per minute
Cuff pressure display range:	.0-299 mm Hg
Memory:	.Automatically stores the last 120 measurements for 2 users (total 240)
Measuring resolution:	.1 mm Hg
Accuracy:	.Pressure within ± 3 mm Hg or 2% of reading >200 mm Hg Pulse $\pm 5\%$ of the reading
Power source:	a) 4 AAA batteries, 1.5 V b) AC adapter 6 V DC 600 mA (voltage 4.5 V DC to 6 V DC)
Accessories:	.Cuff type: Wide range cuff for arm circumference 22-42 cm (8.7"-16.5") Storage case

Technical alterations reserved.

Made in China

11. How to contact us

Microlife USA, Inc.
1617 Gulf to Bay Blvd.
Clearwater, FL 33755

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