Automatic
**Wrist Blood
Pressure
Monitor**

**Instruction
Manual**

FOR MODEL
BP3NP1-3EKRO

Important Product and Safety Information

Follow Instructions for Use. This document provides important product operation and safety information regarding this Blood Pressure Monitor. Please read this document thoroughly before using the device and keep for future reference.

This blood pressure monitor is an automatic digital blood pressure measuring device for use by adults on the wrist at home or in the doctor's office. It enables a very fast and reliable measurement of the systolic and diastolic blood pressure, as well as the pulse rate, by way of the oscillometric method. This device detects the appearance of irregular heartbeats during measurement and provides a warning signal when the irregular heartbeat is detected.

This blood pressure monitor IS intended to be used:

- For self-measurement/monitoring of blood pressure and pulse in adults; whereas the person being measured may be the user/operator of the device.
- Within a home healthcare environment.
- With a cuff located upon the user's wrist; ½ inch from the hand.

This blood pressure monitor IS NOT intended for use with:

- Pregnancy
- Preeclampsia
- Children under 12
- Neonatal patients

WARNING – Potentially hazardous situation that if not avoided may result in serious injury or death.

- Self-measuring means monitoring, not diagnosis or treatment. Unusual values must always be discussed with your doctor. Under no circumstances should you independently alter the dosages of any drugs prescribed by your physician.

- Consult your physician before using this device if any of the following or similar conditions are present: arrhythmias such as atrial or ventricular premature beats or atrial fibrillation, arterial sclerosis, poor perfusion, diabetes, age, pregnancy, preeclampsia, renal diseases. Motion during measurement, including trembling or shivering may affect the measurement.
- Though not for use with children under 12, ensure that any children around this device are supervised; some parts are small enough to be swallowed.

DO NOT:

- ✗ Use this device if you think it is damaged or anything appears unusual (e.g. sporadic operation, open packaging upon purchase).
- ✗ Use the displayed pulse for checking the frequency of heart pacemakers as this device is not suitable for this action.
- ✗ Open/modify this device; inaccuracy and/or harmful injury may result.
- ✗ Conduct many frequent measurements as this may result in harmful injury due to blood flow interference.
- ✗ Place the Cuff over a wound as this may cause further injury.
- ✗ Place and pressurize the Cuff over/near any present intravascular access or therapy, or arteriovenous shunt, as this may cause blood flow interference and result in harmful injury.
- ✗ Place and pressurize the Cuff over a limb near the side of a mastectomy as this may cause harmful injury.
- ✗ Use the Blood Pressure Monitor on a limb simultaneously with other medical equipment on the same limb due to possible interference with such medical equipment.
- ✗ Maintain pressure in the Cuff applied to the limb for a prolonged amount of time. Ensure that circulation in the limb is not impaired by checking circulation if prolonged/repeated exposure to pressure occurs.

- ✗ Service the device when being used or when power is supplied. When the device is not in use and power is removed, batteries may be replaced by the user. No other parts/components are accessible.

CAUTION – Potentially hazardous situation that if not avoided may result in minor/moderate injury, property damage, and/or damage to the device

- This device contains sensitive electronics components. Avoid strong electrical or electromagnetic fields in the direct vicinity of the device (e.g., mobile telephones, microwave ovens). These can lead to temporary impairment of the measuring accuracy. Move the device to another location if interference is determined.
- When not using the Blood Pressure Monitor for extended lengths of time, remove the batteries to avoid potential battery leakage and damage to the monitor.
- When replacing the batteries, ensure all the batteries are replaced simultaneously to avoid battery damage and potential damage to the monitor. Microlife USA does not recommend using rechargeable batteries.
- To avoid inaccurate measurements and to lessen any discomfort from Cuff pressure, ensure the Cuff is placed correctly on the limb and fits correctly when snug (not tight), as indicated by markings with the Cuff.
- Consult your physician in cases of frequent irregular heartbeat detections.
- This Risk Indicator feature is provided in order to help you understand your potential blood pressure risk. However, this feature is neither a diagnosis nor a substitute for a medical examination. It is important to consult with your physician to determine your risk.

DO NOT:

- ✗ Drop this device or expose it to strong vibrations; sensitive components may be affected resulting in inaccuracies and/or operational issues.

- ✗ Use the Blood Pressure Monitor outside of its specified operation temperature and humidity rating, or if stored outside of its specified storage temperature and humidity rating. Avoid storage in direct sunlight.
- ✗ Use this device in a moving vehicle; inaccurate measurements may result.
- ✗ Use third party accessories. Only use Microlife authorized accessories, such as cuffs or AC adapters, as those not approved for use with the device may provide inaccurate measurements, injury, and/or damage the device.

STANDARDS

This medical device is compliant with:

- Medical device and non-invasive blood pressure monitor standards IEC 60601-1, IEC 60601-1-2, IEC 60601-1-11, and AAMI/ANSI/IEC 80601-2-30, and
- Electromagnetic standards IEC 60601-1-2, along with FCC Part 15, and
- Clinical Testing per standard ANSI/AAMI/ISO 81060-2:2013 was conducted on blood pressure device using the same measurement technology.

Please note: According to international standards, your monitor should be checked for accuracy every 2 years.

TYPE

IP20: Protected against solid foreign particles with a diameter of more than 12.5 mm, no protection against water.

Keep Dry. 

Type BF Applied Part

Expected Life

Monitor	5 Years
Cuff	2 Years

Batteries and electronic instruments must be disposed of in accordance with the locally applicable regulations, not with domestic waste.

FCC

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation. Changes or modifications to the product are not approved by Microlife USA and could void the user's authority to operate the equipment under FCC jurisdiction.

This equipment has been verified to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment on and off, the user is encouraged to try to correct the interference by increasing the distance between the product and the affected device; or if applicable, reorientation the receiving antenna, or moving the product's power plug to another receptacle.

Wrist Blood Pressure Monitor

Instruction Manual

Table of contents

1. Introduction

- 1.1. Your Automatic Wrist Blood Pressure Monitor
- 1.2. Inflation Mode Technology

2. How is blood pressure measured?

- 2.1. What is blood pressure, and why does it fluctuate?
- 2.2. Measurement Guidelines
- 2.3. What can I do to change my blood pressure?

3. Components of your blood pressure monitor

4. How do I get started?

- 4.1. Inserting the batteries
- 4.2. Setting the date and time

5. How do I take a measurement?

- 5.1. Preparing to take a measurement
- 5.2. Common errors
- 5.3. Using the cuff
- 5.4. Selecting the user
- 5.5. Taking a measurement
- 5.6. Stopping a measurement
- 5.7. Irregular heartbeat detection
- 5.8. Hypertension risk indicator
- 5.9. Memory: Reviewing readings
- 5.10. Setting the medication reminder

6. Battery change indicator

7. Error messages/troubleshooting

8. Care and maintenance

9. Limited warranty

10. Technical specifications

11. How to contact us

1. Introduction

1.1. Your Automatic Wrist Blood Pressure Monitor

Thank you for purchasing a fully automatic wrist blood pressure monitor. Your monitor is designed to provide fast and reliable digital readings of your pulse, and systolic / diastolic blood pressure using the oscillometric method on your wrist. It offers clinically proven accuracy and has been designed to be user friendly.

Before using your blood pressure monitor, please read this instruction manual carefully to ensure correct use. If you have additional questions regarding blood pressure measurements please contact your doctor.

1.2. Inflation Mode Technology

Please note that this device employs our "Inflation Mode Technology." This means your blood pressure is measured while the wrist cuff is inflating, rather than first inflating to a peak pressure and then measuring during a slow deflation process. This unique technology provides a faster, more comfortable blood pressure reading for you.

If you have any questions about your blood pressure reading or monitor, please call us at 1-800-568-4147.

2. How is blood pressure measured?

2.1. What is blood pressure, and why does it fluctuate?

Your level of blood pressure is determined in the circulatory center of the brain and adjusts to a variety of situations through feedback from the nervous system. To adjust blood pressure, the strength and frequency of the heart (pulse), as well as the width of circulatory blood vessels is altered. Blood vessel width is affected by way of fine muscles in the blood vessel walls. Your level of arterial blood pressure changes periodically during heart activity. During the "blood ejection" (Systole), the value is highest (systolic blood pressure value). At the end of the heart's "rest period"

(Diastole), pressure is lowest (diastolic blood pressure value). Your blood pressure values must lie within certain normal ranges in order to prevent particular diseases.

2.2. Measurement Guidelines

Blood pressure is very high if your systolic blood pressure is over 140 mmHg and/or your diastolic pressure is above 90 mmHg, **while at rest**. In this case, please consult your doctor immediately. Long-term values at this level endanger your health due to continual damage to the blood vessels in your body.

Should the systolic blood pressure values lie between 130 mmHg and 139 mmHg and/or the diastolic blood pressure values lie between 80 mmHg and 89 mmHg, consult your doctor. Regular self-checks will be necessary.

If you have blood pressure values that are too low (i.e., systolic values under 105 mmHg and/or diastolic values under 60 mmHg), consult your physician.

Even with normal blood pressure values, a regular self-check with your blood pressure monitor is recommended. You can detect possible changes in your values early and react appropriately.

If you are undergoing medical treatment to control your blood pressure, keep a record of values along with time of day and date.

Show these values to your physician. **Never use the results of your measurements to independently alter the medication prescribed by your physician.**

Which values are normal?

The following standards for assessing high blood pressure (in adults) have been established by the American Heart Association (AHA) and American College of Cardiology (ACC) in 2017.

Category	Systolic (mm Hg)	Diastolic (mm Hg)
Normal	<120	and <80
Elevated	120-129	and <80
Hypertension		
Stage 1 Hypertension	130-139	or 80-89
Stage 2 Hypertension	140-179	or 90-119
Hypertensive Crisis	≥180	or ≥120

Additional information

- This chart reflects a 2017 update to blood pressure standards.
- If your values are mostly normal under resting conditions but exceptionally high under conditions of physical or psychological stress, it is possible that you are suffering from so-called "labile hypertension." Consult your physician.

2.3. What can I do to change my blood pressure?

- a) Consult your physician.
- b) Increased blood pressure values (various forms of hypertension) are associated with considerable health risks over time. Arterial blood vessels in your body are endangered due to constrictions caused by deposits in the vessel walls (arteriosclerosis). A deficient supply of blood to important organs (heart, brain, muscles) can result. Furthermore, the heart will become structurally damaged with increased blood pressure values.
- c) There are many different causes of high blood pressure. We differentiate between the common primary (essential) hypertension and secondary hypertension. Secondary hypertension can be ascribed to specific organ malfunctions. Please consult your physician for information about the possible origins of your own increased blood pressure values.
- d) There are measures you can take to reduce and even prevent high blood pressure. These measures must be permanent lifestyle changes.

1) Eating habits

- Strive for a normal weight corresponding to your age. See your doctor for your ideal weight.
- Avoid excessive consumption of common salt.
- Avoid fatty foods.

2) Previous illnesses

- Follow consistently any medical instructions for treating illness such as:
 - Diabetes (diabetes mellitus)
 - Fat metabolism disorder
 - Gout

3) Habits


- Give up smoking completely.
- Drink only moderate amounts of alcohol.
- Restrict your caffeine consumption (e.g., coffee).

4) Your physical condition


- **After a medical examination, and with your doctor's approval and direction; exercise.**
- Choose sports which require stamina and avoid those which require strength.
- Avoid reaching the limit of your performance.
- With previous illnesses and/or an age of over 40 years, please consult your doctor before beginning your exercise routine. Speak with your doctor regarding the type and extent of exercise that is appropriate for you.

3. Components of your blood pressure monitor

The illustration shows your blood pressure monitor, consisting of:

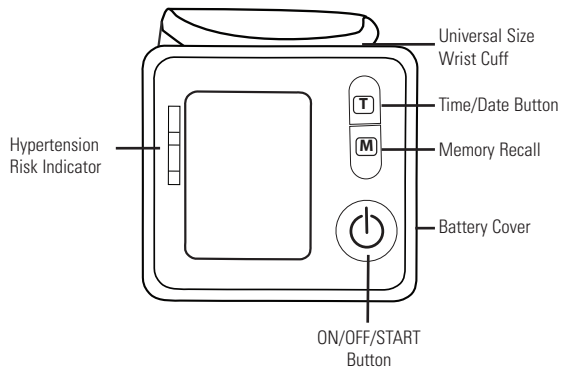Fits wrist size 13.5-19.5 cm (5.3"-7.7")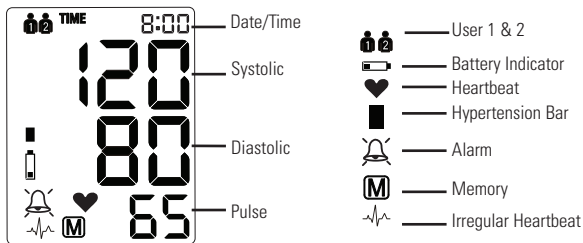

4. How do I get started?

4.1. Inserting the batteries

After you have unpacked your device, insert the batteries. The battery compartment is located on the underside of the device.


- Remove the battery cover.
- Insert the batteries (2 x size AAA 1.5 V), following the indicated polarity.
- If the battery warning appears in the display, the batteries are discharged and must be replaced with new ones.

Attention!

- Please use AAA Long-Life or Alkaline 1.5 V batteries and replace them all at the same time.

4.2. Setting the date and time


- After the new batteries are activated, the year number flashes in the display. You can advance the year by pressing the memory or "M" button. To confirm and then set the month, press the time or "T" button.
- You can now set the month using the "M" button. To confirm and then set the day, press the "T" button.
- Please follow the instructions above to set the day, hour and minutes.
- Once you have set the last minute and pressed the "T" button, the date and time are set and the time is displayed.
- If you want to change the date and time, press and hold the "T" button down for approx. 3 seconds until the year starts to flash. Now enter the new values as described above.

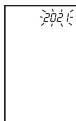Memory
Button

Time/Date
Button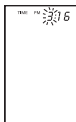

5. How do I take a measurement?

Please note: You should always be seated before and during the measurement.

5.1. Preparing to take a measurement

- Avoid eating, smoking as well as all forms of exertion directly before your measurement. These factors influence the measurement result. Relax by sitting in an armchair in a quiet atmosphere for about 5 minutes before your measurement.
- Always measure on the same wrist (normally left).
- Compare readings at the same time of day, since your blood pressure changes during the course of the day.

5.2. Common errors


Note:

Comparable blood pressure measurements always require the same conditions. These are normally always quiet conditions.

- All efforts to support the arm can increase blood pressure. Make sure you are in a comfortable, relaxed position and do not flex any of the muscles in the measurement arm during the measurement. Use a cushion for support if necessary.
- If the wrist artery lies considerably lower (higher) than the heart, an erroneously higher (lower) blood pressure will be measured. (Each 15 cm/6" difference in height results in a measurement error of 10 mmHg.)
- A loose cuff causes false measurement values.
- With repeated measurements, blood can accumulate in the respective arm, which can lead to false results. Correctly executed blood pressure measurements should therefore first be repeated after a 1 minute pause in order to allow the accumulated blood to flow away.

5.3. Using the cuff

- a) Remove all objects and jewelry (e.g., wristwatch) from the wrist in use. This is normally the left wrist. Put the cuff over the wrist with the monitor face on the inside of the wrist.
- b) The distance between the cuff and the hand should be approx. 10 mm (1/2").
- c) Secure the cuff with the hook and loop fastener so that it lies comfortably and not too tight. No space should remain between the cuff and the wrist.
- d) Support the arm with the monitor case so that the cuff rests at the same height as the heart. Remain still for 5 minutes before beginning the measurement.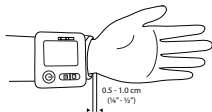

5.4. Selecting the user

With the unit powered off:


- Press the TIME button to toggle between user 1 and user 2.
- Press the MEMORY button to confirm your user selection.

5.5. Taking a measurement

After the cuff has been appropriately positioned the measurement can begin. Remain still during your measurement, do not flex muscles.

- Remain seated quietly for 5 minutes before you begin the measurement.
- Press the POWER button to begin. After pressing the POWER button you will hear a series of short beeps and the pump begins to inflate the cuff. The measurement is performed while cuff inflates. The pumping speed can vary, which is completely normal. The increasing cuff pressure will show on the display.
- When the device has detected the pulse, the heart symbol in the display begins to blink.
- When the measurement has been concluded, air in the cuff is released. The measured systolic and diastolic blood pressure values as well as the pulse are now displayed. (Pulse=number of heartbeats each minute) Example (Fig.): Systolic 118, Diastolic 73, Pulse 75. The measurement results are displayed for about one minute.

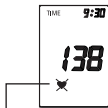Pulse Detected -
Unit measuring


5.6. Stopping a measurement

If it is necessary to interrupt a blood pressure measurement for any reason (e.g., the patient feels unwell), the ON/OFF button can be pressed at any time. The device then immediately lowers the cuff pressure automatically and enters into sleep mode.

5.7. Irregular heartbeat detection

This symbol  indicates that certain pulse irregularities were detected during the measurement. In this case, the result may deviate from your normal basal blood pressure – repeat the measurement. In most cases, this is no cause for concern. However, if the symbol appears on a regular basis (e.g., several times a week with measurements taken daily), we advise you to tell your doctor.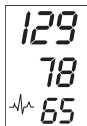

Please show your doctor the following explanation:

Information for the doctor on frequent appearance of the irregular heartbeat symbol

This instrument is an oscillometric blood pressure monitor device that also analyzes pulse frequency during measurement. The instrument is clinically tested.


If pulse irregularities occur during measurement, the irregular heartbeat symbol is displayed after the measurement. If the symbol appears more frequently (e.g., several times per week on measurements performed daily) or if it suddenly appears more often than usual, we recommend the patient to seek medical advice.

The instrument does not replace a cardiac examination but serves to detect pulse irregularities at an early stage.

5.8. Hypertension risk indicator

The colored indicator on the left-hand edge of the display has been designed to provide a quick visual representation of your blood pressure. Once a measurement has been completed, a black bar will display onscreen next to the colored hypertension indicator. The height of the black bar will show if the measurement is within the normal (green), borderline (yellow/orange) or danger (red) range.

This classification is based on standards established by the American Heart Association (AHA) and American College of Cardiology (ACC) in 2017.

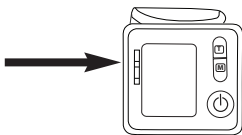If the black bar is in the:

- **green zone**, your measurement is "Normal."
- **lower yellow zone**, it is "Elevated."
- **upper yellow zone**, it is "Stage 1 Hypertensive."
- **orange zone**, it is "Stage 2 Hypertensive."
- **lower red zone**, it is "Stage 2 Hypertensive."
- **upper red zone**, it is "Hypertensive Crisis."

Indication of a "Normal" Blood Pressure	Indication of an "Elevated" Blood Pressure	Indication of "Stage 1 Hypertensive"	Indication of a "Stage 2 Hypertensive"	Indication of a "Stage 2 Hypertensive"	Indication of a "Hypertensive Crisis"
Red	Red	Red	Red	Red	Red
Red	Red	Red	Red	Red	Red
Orange	Orange	Orange	Orange	Orange	Orange
Yellow	Yellow	Yellow	Yellow	Yellow	Yellow
Yellow	Yellow	Yellow	Yellow	Yellow	Yellow
Green	Green	Green	Green	Green	Green

5.9. Memory: Reviewing readings

At the end of a measurement, this monitor automatically stores each result with date and time. This unit stores 99 memories for each of 2 users.

Viewing the stored values

With the unit off (or with only the time and user icons showing), press the "M" button. The display first shows "A," then shows an average of all measurements stored in the unit. Please note: Measurements for each user are averaged and stored separately. Be certain that you are viewing the measurements for the correct user.

Pressing the "M" button again displays the previous value. To view a particular stored memory, press and hold the "M" button to scroll to that stored reading.

Memory full

When the memory has stored 99 results, a new, measured value is stored by overwriting the oldest value.

Clear all values

If you are sure that you want to permanently remove all stored values, hold down the "M" button (the instrument must have been switched off beforehand) until "CL" appears and then release the button. If you do not want to clear the values, press the POWER button. To permanently clear the memory, press the "M" button while "CL" is flashing. Individual values cannot be cleared.

5.10. Setting the medication reminder


This instrument allows you to set two alarm times at which an alarm signal will then be triggered. This can be a useful aid, for instance as a reminder to take medication or to remind you to take your blood pressure at the same time each day.

1. To set an alarm time, press the time button (the instrument must have been switched off beforehand) and immediately afterwards the "M" button and hold both down until the bell symbol appears in the bottom left of the display. Then release both buttons. The flashing "1" in the display indicates that the first alarm time can now be set.
2. Press the time button to set the hours – the hours display flashes and pressing the "M" button allows you to set the alarm hour. To confirm, press the time button.
3. The minute display will now flash. The minutes can be set using the "M" button. To confirm, press the time button again.
4. The bell symbol will now flash. Use the "M" button to select whether the alarm time is to be active (bell) or inactive (crossed-out bell). To confirm, press the time button.

To set a second alarm time, proceed as above but if the "1" flashes, press the "M" button to select "2" and confirm with the time button.

- An active alarm time is indicated by the bell symbol in the display.
- The alarm sounds at the set time every day.
- To switch-off the alarm when it is sounding, press the START/STOP button.
- To permanently switch-off the alarm, proceed as above (steps 1-4) and select the crossed-out bell symbol. This will then disappear from the display.
- The alarm times must be re-entered each time the batteries are replaced.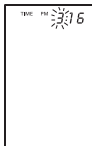

6. Battery change indicator

Batteries almost discharged

When the batteries are approximately 75% used, the battery symbol will flash as soon as the instrument is switched on (if at least one of the batteries still has some charge). Although the instrument will continue to measure reliably, you should obtain replacement batteries.

Batteries discharged – replacements required

When the batteries are discharged, the battery symbol will appear unblinking as soon as the instrument is switched on. You cannot take any additional measurements and must replace the batteries.

1. Open the battery compartment at the bottom of the instrument.
2. Replace all of the batteries at the same time – ensure they are inserted correctly, as shown on the symbols in the compartment.
3. The memory retains all values although date and time must be reset – the year number will flash automatically after batteries are replaced.
4. To set date and time, follow the procedure described in Section 4.2.

Note:

Use two new, Long-Life 1.5 V AAA batteries. Do not use batteries beyond their expiration date. If the monitor is not going to be used for a prolonged period, the batteries should be removed.

7. Error messages/troubleshooting

If an error occurs during a measurement, the measurement is discontinued and a corresponding error code is displayed (Example: Error No. 2).

Error No.	Possible cause(s)
ERR 1	No pulse was detected.* Ensure the cuff is properly placed on the inside of your wrist and repeat a measurement.
ERR 2	Unnatural pressure impulses influenced the measurement result. Reason: The arm was moved during the measurement (artifact). Repeat, keeping still and quiet.
ERR 3	Inflation of the cuff takes too long. The cuff is not correctly seated. Re-position cuff and repeat the measurement.
ERR 5	The measured readings indicated an unacceptable difference between systolic and diastolic pressures. Take another reading following directions carefully. Contact your doctor if you continue to get unusual readings.
HI	The cuff pressure is too high. Relax for 5 minutes and repeat the measurement.*
LO	The pulse is too low (less than 40). Repeat the measurement.*

*** If this or any other problem occurs repeatedly, please consult your doctor.**

Other possible malfunctions and solutions

If problems occur when using the device, the following points should be checked and, if necessary, the corresponding measures are to be taken:

Malfunction	Remedy
The display remains blank when the instrument is switched on.	<ol style="list-style-type: none"><li data-bbox="441 246 977 284">1. Check batteries for correct polarity.<li data-bbox="441 291 977 375">2. If the display is unusual, re-insert batteries or exchange them.
The device frequently fails to measure blood pressure values, or the values measured are too low (too high).	<ol style="list-style-type: none"><li data-bbox="441 382 977 420">1. Check the positioning of the cuff.<li data-bbox="441 427 977 589">2. Measure blood pressure again in peace and quiet, carefully following the details in Section 5.
Every measurement produces a different value.	<p data-bbox="441 596 977 715">Please read the following information listed in Section 5.2 "Common sources of error." Repeat the measurement.</p> <p data-bbox="441 722 977 848">Please note: Blood pressure fluctuates continually so successive measurements will show some variability.</p>
Blood pressure values measured at your doctor's office differ from your measurements.	<p data-bbox="441 855 977 932">Record the daily development of the measured values and consult your doctor.</p> <p data-bbox="441 939 977 1100">Please note: Individuals visiting their doctor frequently experience anxiety which can result in a higher reading than at home under resting conditions.</p>

8. Care and maintenance

- a) The cuff contains a sensitive airtight bubble. Handle this carefully and avoid all types of straining through twisting or buckling.
- b) Clean the device with a soft, dry cloth. Do not use gasoline, thinners or similar solvents. Spots on the cuff can be removed carefully with a damp cloth and soapsuds. The cuff must not be washed
- c) Never open the monitor. This voids the manufacturer's warranty.

9. Limited warranty

Your Wrist Blood Pressure Monitor is warranted for the estimated life of the monitor by Microlife USA Inc, against manufacturer defects for the original purchaser only, from date of purchase.

The lifetime warranty applies to the monitor only. Batteries are not covered by this warranty.

The warranty does not apply to consequential and incidental damages, or damage caused by batteries, improper handling, and accidents. Professional use, not following the operating instructions, and alterations made to the monitor or accessory by third parties, are also not included in this warranty. Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitation or exclusion may not apply to you.

Microlife USA Inc will investigate your concern. A Monitor or Accessory as defined by this warranty, determined to be out of specification, will be replaced and shipped to you at no cost. A Monitor or Accessory as defined by this warranty, determined to be within specification, will be returned to you with a report of findings, at no cost.

Please use the below customer service contact information to reach Microlife USA Inc. regarding any warranty concerns. We ask that you please contact us before sending any product back in order to better identify, and more quickly process, your concern.

This warranty gives you specific legal rights, and you may also have other rights which vary from state to state.

10. Technical specifications

Weight:	138 g (with batteries)
Size:	81 x 65 x 28 mm
Storage temperature:	-20 to +55°C (-4 to +131°F)
Humidity:	15 to 90% relative humidity non condensed
Operation temperature:	10 to 40°C (50 to 104°F)
Display:	LCD (Liquid Crystal Display)
Measuring method:	Oscillometric
Pressure sensor:	Capacitive
Measuring range:	
SYS:	60 to 255 mmHg
DIA:	40 to 200 mmHg
Pulse:	40 to 200 per minute
Cuff pressure display range:	0-299 mmHg
Memory:	Automatically stores the last 99 measurements
Measuring resolution:	1 mmHg
Accuracy:	Pressure within ± 3 mmHg or 2% of reading >200 mmHg Pulse $\pm 5\%$ of the reading
Power source:	2 dry cells (batteries) Size AAA 1.5 V

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